SBK without compromise using Moria One Use-Plus



One Use-Plus by numbers



2+ million patients operated



300,000+ patients treated / year since last 3 years



120,000+ eyes followed in peer-reviewed protocols



45+ peer-reviewed published studies






SBK without compromise to reach ...



Flap creation in less than 4 seconds for a minimal suction time, potentially preserving goblet cells¹

- with a full intraoperative visibility during the whole flap creation

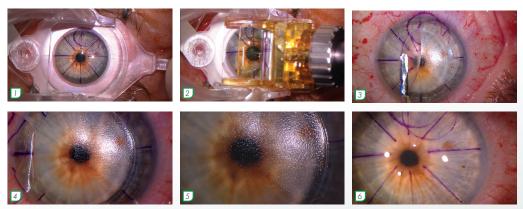

Sequence showing flap creation and lifting on a patient's right eye using Moria One Use-Plus SBK Courtesy of James S. Lewis, MD (Elkins Park, PA, USA)



Predictible thin sub-Bowman flaps centrally²⁻¹²

- with a high reproducibility between OD then OS^{2-4,8,12}
- with a planar profile/architecture throughout the whole flap surface⁵⁻⁷





Anterior segment OCT pictures of a nasal-hinged corneal flap profile created using Moria One Use-Plus SBK Courtesy of James S. Lewis, MD (Elkins Park, PA, USA)


When using the SBK 90-µm calibrated head, mean central flap thickness is:

- at Speed 2 (fast motion): 100 ± 10 [80 120] microns
- at Speed 1 (slow motion): $110 \pm 10 [90 130]$ microns



Excellent stromal surface smoothness for an accurate photoablation

- confirmed by Scanning Electron Microscopy¹³
- no « velcro-type » surface for a crystal clear anatomical flap apposition^{6,13-14}

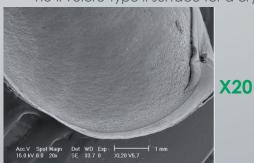

Ac V Spot Maph Diet WD Esp. 200 jmn 15 0 JV 5 0 2160 v. St. 341 0 3420 v5 2

X160

Scanning Electron Microscopy pictures at different magnifications after cutting a flap with Moria One Use-Plus SBK with an intended flap thickness of 100 microns

Courtesy of Richard J. Duffey, MD (Mobile, AL, USA)¹³

... highly-expected patient satisfaction



Excellent safety profile:

- intraoperatively:
 - no buttonhole, no incomplete flap, no epithelial erosion, no irregular stromal bed reported in a large-scale retrospective study on flat corneas (2883 eyes)¹⁴
 - with very large flaps (>9.5 mm) required for high hyperopia and presbyopia corrections¹⁵

postoperatively:

- no single flap displacement¹⁴, no haze nor energy-related DLK¹⁶
- one of the lowest incidence rate of epithelial ingrowth (0.49%) in a large-scale cohort $study^{17}$
- one of the lowest myopic & hyperopic LASIK retreatment rates in large-scale cohort studies: <0.5% & 4.6% respectively¹⁸⁻¹⁹



Very fast visual recovery:

- equivalent to Femto-LASIK^{1,11}
- providing a real « WoW » effect far awaited by every patient




Excellent quality of vision:

- limited changes of higher order aberrations (HOAs)9,20-22
- and evenly less induction of total HOAs and spherical aberrations than during Femto- $\rm I\,ASIK^{23}$
- intraocular straylight measurements equivalent to Femto-LASIK²⁴
- negligible changes of higher order Point Spread Function (PSF) Strehl ratio, preserving an optimal retinal image quality²¹




In terms of **in-vivo confocal microscopy** research comparing SBK to Femto-LASIK and other microkeratome⁶:

- higher density of stromal keratocytes at 3 months postoperatively
- faster regenerative velocity of subbasal nerve fibers due to the nasal hinge which preserves more anatomically-placed corneal nerve branches.

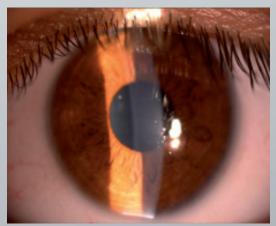

Slit-lamp picture of a SBK flap at Day 1 postoperative: edges are almost invisible. Courtesy of Ahmed El-Massry (Alexandria, Egypt)

What makes One Use-Plus microkeratome the stateof-the-art in today's automated microkeratomes

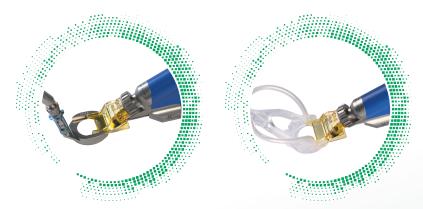

Excellent ergonomics for a user-friendly automated microkeratome



- ✓ Automated linear motion to benefit from nasal hinge^{3,7,10,12,17,19}
- ✓ Pre-assembled and one-handed usage possible
- ✓ Safety and reliability of two independent motors: one for head advancement, one for blade oscillation



- ✓ Wide range of suction rings to individualize flap geometry based on photoablative patterns: from oval-shaped to extra-large hyperopic treatments¹⁵
- ✓ Adjustable stops for customized hinge length
- ✓ Design of suction ring makes the use of a speculum unnecessary on small fissures
- ✓ Translucent single-use plastic ring enables visual confirmation of suction



Unique choice of single-use calibrated cutting heads & suction rings & aspiration tubings

Single-use means simplicity, safety, convenience, and ease-of-use:

- ✓ Protected blade to avoid potential damage
- ✓ Limits complications and risks linked to damaged or improperly maintained reusable heads
- ✓ Unique solution to limit infection and contamination from prions, virus, bacteria, germs and other micro-organisms²⁵⁻²⁷



And as a bonus:

- ✓ Single-use heads and rings facilitate compliance with ASCRS guidelines which recommend not using flash sterilization
- ✓ Eliminates sterilization and maintenance
- ✓ Lower initial investment costs
- ✓ More rapid patient turnover, leading to greater efficiency



The most economical platform for SBK:

- ✓ Limited capital investment and cost per surgery
- ✓ Make your own comparisons between the One Use-Plus and a femtosecond laser in terms of capital investment, disposables per patient, and annual maintenance.

Product references



One Use-*Plus* – heads 90/130 Box of 10 units





19393/90 19393/130

One Use-Plus: metallic reusable suction rings



19391/-1 19391/0



19391/1 19391/2 19391/3



19519/-1



19391/1/OV 19391/2/OV 19391/3/OV

One Use-*Plus*: single-use plastic suction rings & heads

Box of 10 units



19336/XX 19337/XX 19354/XXX

XX = 90 - 130XXX = 110 - 130

	#19345	#22519514	#22519513	#19391/0	#19391/1	#19391/2	#19391/3
SETM	✓	1	✓	-	-	-	-
SETM9	✓	✓	✓	✓	√	✓	1

SETM: ONE USE-PLUS Set

Set containing the One Use-Plus microkeratome for LASIK surgery:

• One Use-Plus handpiece: #19345

• storage box for One Use-Plus microkeratome: #22519514

• sterilization box for One Use-Plus suction rings: #22519513

SETM9: ONE USE-PLUS Set (Metal Rings)

Set containing the One Use-*Plus* microkeratome for LASIK surgery with 4 reusable metallic suction rings:

• One Use-Plus handpiece: #19345

• storage box for One Use-Plus microkeratome: #22519514

• sterilization box for One Use-Plus suction rings: #22519513

• One Use-*Plus* reusable suction ring, size 0: #19391/0

• One Use-Plus reusable suction ring, size +1: #19391/1

• One Use-Plus reusable suction ring, size +2: #19391/2

• One Use-Plus reusable suction ring, size +3: #19391/3

Chen et al. Anterior segment optical coherence tomography measurement of flap thickness after myopic LASIK using the Moria One Use-Plus microkeratome.

J Refract Surg. 2010;26(6):403-410.

- 3. Lian et al. Correlation analysis and corneal flap thickness changes among Moria SBK, 90 and 110 microkeratome in LASIK. Chin Ophthalmic Res. 2010;28(12):1158-1161.
- Du et al. Flap thickness variation with 3 types of microkeratome heads. J Cataract Refract Surg. 2011;37(1):144-148.
- Sun et al. Comparisons of morphologic characteristics between thin-flap LASIK and SBK. Int J Ophthalmol. 2012;5(2):338-342.
- 6. Zhang et al. Confocal comparison of corneal nerve regeneration and keratocyte reaction between FS-LASIK, OUP-SBK, and conventional LASIK.

Invest Ophthalmol Vis Sci. 2012;53:5536-5544.

- 7. Zhai et al. Comparison of the flaps made by femtosecond laser and automated keratomes for Sub-Bowman Keratomileusis. Chin Med J. 2013;126(13):2440-2444.
- 8. Zhang et al. Comparison of corneal flap thickness using a FS200 femtosecond laser and a Moria SBK microkeratome. Int J Ophthalmol. 2014;7(2):273-277.
- 9. Al-Thomali TA. Reproducibility of flap thickness in Sub-Bowman Keratomileusis using a mechanical microkeratome. J Cataract Refract Surg. 2014;40(11):1828-1833.
- 10. Mimouni et al. Factors affecting laser in situ keratomileusis flap thickness: comparison of 2 microkeratome heads. J Cataract Refract Surg. 2015; 41 (2):348-353.
- 11. Xu et al. The impact of flap creation methods for Sub-Bowman's Keratomileusis (SBK) on the central thickness of Bowman's layer. PLoS ONE 2015;10(5):e0124996.
- 12. Katz et al. Flap-induced astigmatism in eyes with sphere myopia correction: superior hinge using a rotating microkeratome versus nasal hinge using a linear microkeratome.

 J Cataract Refract Surg. 2015;41(6):1160-1167.
- 13. Duffey RJ. Moria One Use-*Plus* SBK microkeratome: predictably thin, smooth, planar flaps for faster visual recovery. Paper presented during the 26th annual meeting of European Society of Cataract and Refractive Surgery; Sept 13-17, 2008; Berlin, Germany.
- **14.** Falcon et al. Safety of the automated microkeratome for Sub-Bowman's Keratomileusis on the flat cornea. French J Ophthalmol. 2016;39(2):202-209.
- 15. Gauthier-Fournet et al. Six-month outcomes after high hyperopia correction using Laser-Assisted In Situ Keratomileusis with a large ablation zone.

Cornea 2019;38(9):1147-1153.

- 16. Kasetsuwan et al. Comparison of performances of femtosecond laser and microkeratome for thin-flap Laser In Situ Keratomileusis. Lasers Surg Med. 2016;48(6):596-601.
- Friehmann et al. Risk factors for epithelial ingrowth following microkeratome-assisted LASIK. J Refract Surg. 2018;34(2):100-105.
- **18.** Pokroy et al. Myopic laser in situ keratomileusis retreatment: incidence and associations. J Cataract Refract Surg. 2016;42(10):1408-1414.
- 19. Mimouni et al. Risk factors for re-treatment following hyperopic LASIK. J Refract Surg. 2018;34(5):316-320.
- 20. Hassanin et al. Changes in corneal wavefront aberrations and asphericity following optimized LASIK ablation in moderate to highly myopic eyes

J Applied Sci Res. 2013;9(3):2404-2410.

- 21. McAlinden et al. Comparison of Higher Order Aberrations after LASIK and LASEK for myopia. J Refract Surg. 2010;26(1):45-51.
- 22. McAlinden et al. The change in internal aberrations following myopic corneal laser refractive surgery. Graefes Arch Clin Exp Ophthalmol. 2011;249(5):775-781.
- 23. Malhotra et al. Higher Order Aberrations and visual outcomes in Wavefront-Optimized Sub-Bowman Keratomileusis: flap creation using femtosecond laser versus mechanical microkeratome. Asia Pac J Ophthalmol. 2015;4(4):197-203.
- 24. Wang et al. Intraocular straylight after thin-flap LASIK with a femtosecond laser versus a mechanical microkeratome. J Refract Surg. 2013;29(8):534-539.
- **25.** Orrù et al. Prion seeds distribute throughout the eyes of sporadic Creutzfeldt-Jakob disease patients. *mBio* 2018:9:e02095-18.
- 26. Chen et al. Ocular manifestations of a hospitalised patient with confirmed 2019 novel coronavirus disease. Br J Ophthalmol. 2020;104(6):748-751.
- 27. Kumar et al. Presence of viral RNA of SARS-CoV-2 in conjunctival swab specimens of COVID-19 patients. *Indian J Ophthalmol.* 2020;68(6):1015-1017.

To obtain more information

15, rue Georges Besse 92160 Antony FRANCE Phone: +33 (0) 1 46 74 46 74 Fax: +33 (0) 1 46 74 46 00 www.moria-surgical.com

MORIA SA

1050 Cross Keys Drive Doylestown, PA 18902 USA Phone: (800) 441 1314 Fax: + 1 (215) 230 7670 **Email: orders@moriausa.com**

MORIA Inc

Moria Japan K.K. Arcadia Building 6F 1-12-3 Kanda SudachoChiyoda-Ku Tokyo 101-0041 JAPAN Phone: 81-3-6260-8309

Fax: 81-3-6260-8310

www.moriajapan.com

Moria Commercial (CHINA) CO., LTD.
上海目利亚贸易有限公司
Room H 6F Kaili Building NO.432
West Huai Hai Road, Changning district, Shanghai, 200052, P.R.C
中国上海市长宁区淮海西路432号
凯利大厦6楼08室
Tel/Fax: +86 021 52586095
www.moria-surgical.com







MORIA #66065EN-D-10.2020