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
User instructions for the single-use toric IOL pre-op marker, with bubble level #17249

The Moria marker with bubble level is designed for pre-op use in cataract surgery to determine the location of the horizontal meridian. The marks made serve as a reference for positioning of a toric IOL.


- Internal diameter = 9 mm
- External diameter = 14 mm
- Ⓐ: 2 marking blades of 2.5 mm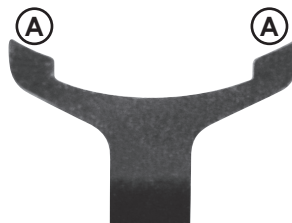

Usage instructions:

Do not use the marker if it has been damaged, leaks, or has fissures of any size.

- 1- Sit the patient down with her/his legs dangling in front, head straight. The patient must stare at a point directly in front of her/him.
- 2- Ink the marking blades with an ink suitable for eye surgery.
- 3- Move the marker toward to the patient's eye, handle facing downwards; the marking blades must be in front of the patient's 180-degree horizontal meridian.
- 4- Adjust the orientation of the marker if necessary; the bubble must be between the 2 lines. When the bubble is between the marks, the precision is within 1°.
- 5- Mark the eye.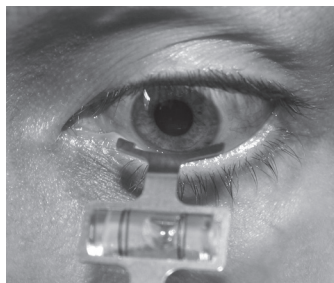

- 6- If necessary, retrace the marks obtained using a pen suitable for eye surgery so the marks remain more visible.
After use, dispose of the marker in the appropriate container for destruction (according to regulations in force).

To obtain more information

MORIA SA
27, rue du Pied de Fourche
03160 Bourbon l'Archambault
FRANCE
Phone: +33 (0) 4 70 67 09 04
Fax: +33 (0) 4 70 67 06 61
www.moria-surgical.com

MORIA Inc
1050 Cross Keys Drive
Doylstown, PA 18902
USA
Phone: (800) 441 1314
Fax: +1 (215) 230 7670
www.moria-surgical.com

MORIA JAPAN KK
Inami Bldg, 6F
3-24-2 Hongo Bunkyo-ku
Tokyo 113-0033
JAPAN
Phone: +81 3 5842 6761
Fax: +81 3 5842 6762
www.moriajapan.com

MORIA in CHINA
Moria Commercial (China) Co. Ltd
Rm H, L, 6FL, Kai Li Building
N° 432 Huai Hai West Road
Shanghai 200052
CHINA
Phone: +86 21 5258 5066
Fax: +86 21 5258 5067
www.moria-surgical.com.cn

Document Aletiq : Rév. D-05.2023 (Validée)