


Hand-Held  
Instruments

## CAN Pendular Marker

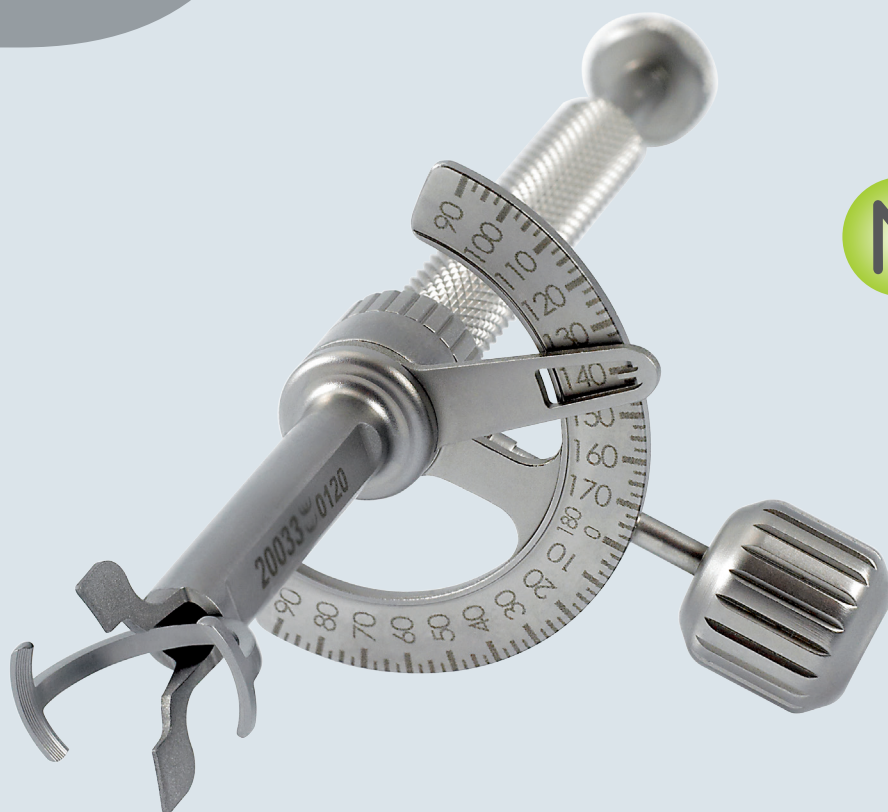


New


## The One-Step Marker

This instrument aims to mark the astigmatism axis in one single step to ensure correct orientation of the toric implant in the anterior chamber.


## #20033 CAN Pendular Marker in details

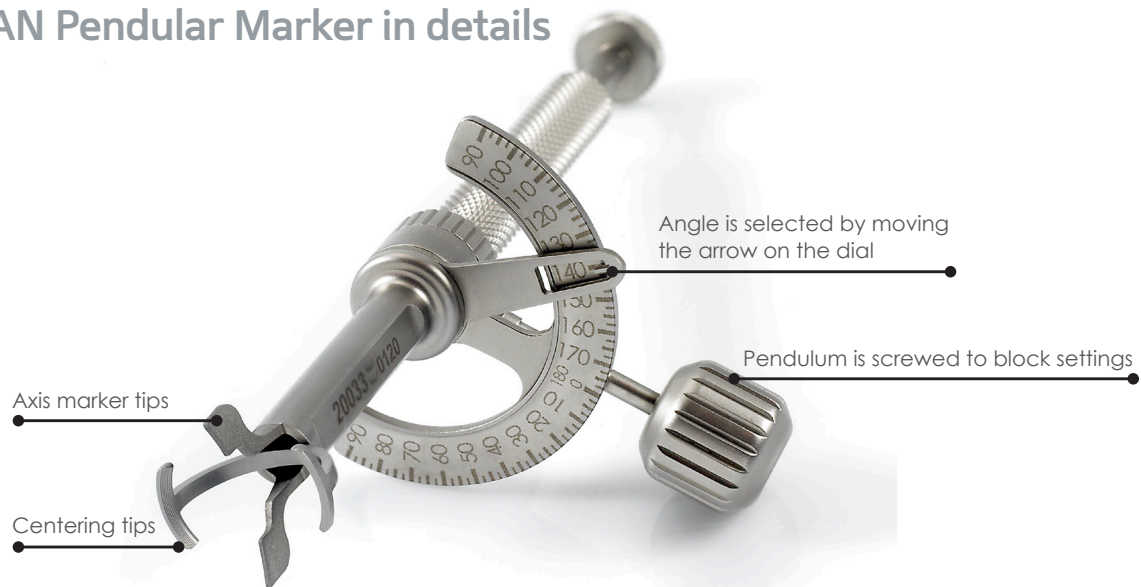

The success of toric IOLs implantation depends on correct measurements and marking. The one-step Can marker helps the surgeon in this procedure.

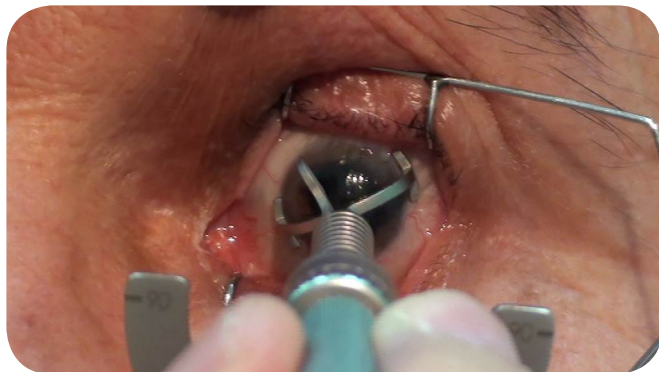

The limbus centering tips provide a support to the surgeon to prevent vibrating in order to make a comfortable marking. Moreover, centering the instrument prevents vertical and horizontal marking errors.

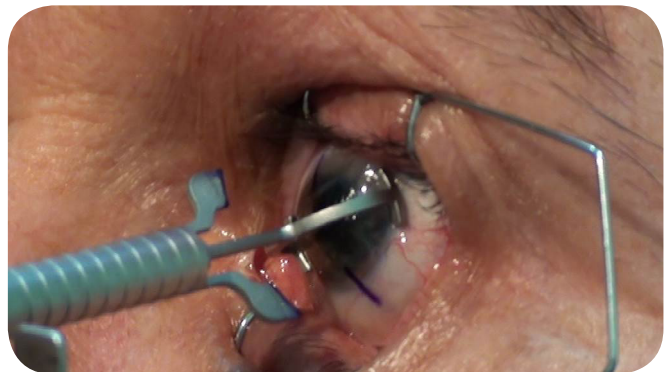

Marking shows intended alignment axis. This will allow the surgeon to rotate the IOL implant until the IOL markings match with the alignments marks on the cornea.


There are several methods to align the toric IOL at the intended axis, most methods follow a 3-step procedure<sup>1</sup>. I worked on this design in order to simplify marking while keeping it accurate.

### Dr CAN gives following benefits:

- No need for additional intraoperative instruments.
- Additional marking errors with a 3-step procedure are eliminated.
- Aligning centering tips to the limbus prevents vertical and horizontal errors.
- It is user-friendly and easy to use with both right and left eye.



Dr Ertugrul Can  
Ondokuz Mayıs Universitesi, Tıp Fakultesi - Samsun - TURKEY

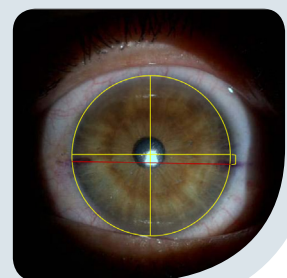

1. Visser N, Berendschot TJM, Bauer NJC, Jurich J, Kersting O, Nuijts RMMA. Accuracy of toric intraocular lens implantation in cataract and refractive surgery. J Cataract Refract Surg. 2011;37(8):1394-1402.

To obtain more information

**MORIA SA**  
15, rue Georges Besse  
92160 Antony  
FRANCE  
Phone: +33 (0) 1 46 74 46 74  
Fax: +33 (0) 1 46 74 46 70  
[www.moria-surgical.com](http://www.moria-surgical.com)

**MORIA Inc**  
1050 Cross Keys Drive  
Doylestown, PA 18902  
USA  
Phone: (800) 441 1314  
Fax: +1 (215) 230 7670  
[www.moria-surgical.com](http://www.moria-surgical.com)

**MORIA JAPAN KK**  
Inami Bldg. 6F  
3-24-2 Hongo Bunkyo-ku  
Tokyo 113-0033  
JAPAN  
Phone: +81 3 5842 6761  
Fax: +81 3 5842 6762  
[www.moriajapan.com](http://www.moriajapan.com)

**Moria Shanghai Office**  
RM4, 10 FL, Kai Li Building  
N°432 Huai Hai West Road  
200052 Shanghai  
CHINA  
Phone: +86 21 5258 5068  
Fax: +86 21 5258 5067  
[www.moria-surgical.com.cn](http://www.moria-surgical.com.cn)

**Moria**  
Leading Innovations in Ophthalmology