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Anterior Segment Surgery

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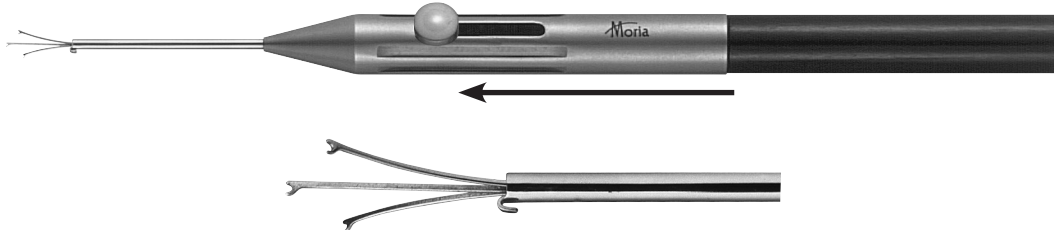
Use and maintenance of Beehler pupil dilators

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
Cleaning and disinfection instructions

- 1** Sortir les branches au maximum en poussant le bouton latéral.
Extend the prongs to the maximum position by sliding the thumb button forward.

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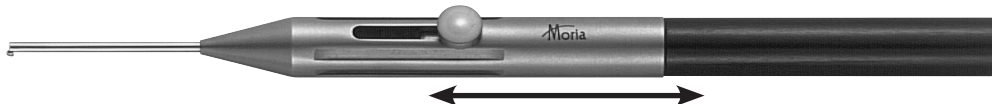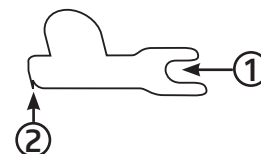
FOLLOWING THESE RECOMMENDATIONS WILL HELP YOU MAINTAIN THE QUALITY OF YOUR INSTRUMENT. IMPROPER MAINTENANCE AND STORAGE MAY DAMAGE IT.

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The Beehler pupil dilator is supplied with a locking device for use during transport. Remove the device before use. To lock: place the notch (1) of the jumper against the knob and clip the end (2) in the top of the slit. To remove it, proceed in the reverse order.

THIS PROTECTION IS ESSENTIAL FOR YOUR INSTRUMENT.

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Cleaning, disinfection and sterilization protocols are available in the Member Area of the Moria website at www.moria-surgical.com. Username and Password required to obtain access.

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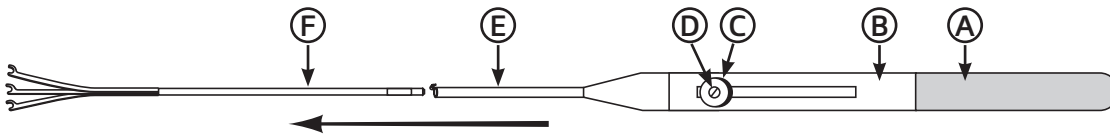
The instruments are delivered with a Teflon® and silicone protection on the tip. This protective cap must stay permanently on the instrument (including during the sterilization and the storage). The protection must be delicately removed before to be used and replaced immediately after cleaning.

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
Instructions for the replacement of the hooks

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Slide button C as far as possible toward the prongs.
Unscrew part D with a screwdriver. Do not unscrew completely.
Remove part F by pulling it gently.
Clean inside tube E with water. Dry with filtered compressed air.

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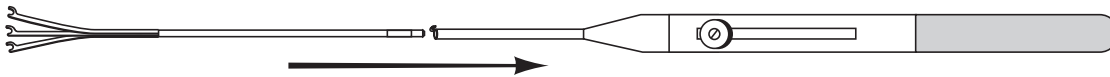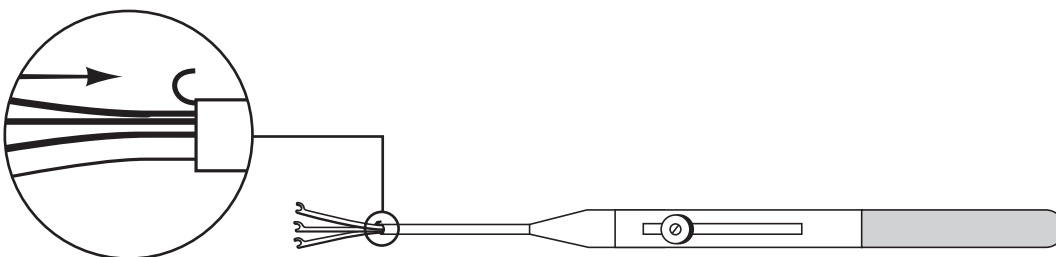
Firmly hold button C and slide part F until the mark is aligned with the extremity of tube E.
Block screw D.
Slide button C back and forth to check that it functions properly.

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MORIA Inc
1050 Cross Keys Drive
Doylestown, PA 18902
USA
Phone: (800) 441 1314
Fax: +1 (215) 230 7670
www.moria-surgical.com

MORIA JAPAN KK
Inami Bldg. 6F
3-24-2 Hongo Bunkyo-ku
Tokyo 113-0033
JAPAN
Phone: +81 3 5842 6761
Fax: +81 3 5842 6762
www.moriajapan.com

Moria Shanghai Office
RM4, 10 FL, Kai Li Building
N°432 Huai Hai West Road
200052 Shanghai
CHINA
Phone: +86 21 5258 5068
Fax: +86 21 5258 5067
www.moria-surgical.com.cn