

# Moria Range of Products



LASIK  
Surgery




Epi-LASIK  
Surgery



Corneal  
Transplant



Evolution  
3E

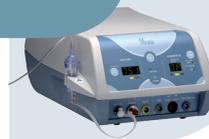

This document is not for use in USA

**Moria**  
Leading Innovations in Ophthalmology

# LASIK Surgery

## Intelligently designed

- Safety and reliability of two independent motors:
  - one for head advancement,
  - one for blade oscillation.
- Simplicity of pre-set parameters: suction time, oscillation speed, advancement rate.
- Customization of flap thickness, diameter, and hinge
- Large Cut solutions for hyperopes, flat corneas, wavefront-guided ablations and lasers requiring large ablation zones characteristics.

## The One Use-*Plus* System

**One Use-*Plus* SBK offers accuracy, predictability and reproductibility equivalent to femto SBK, with faster recovery and at a fraction of the cost.**



- Pre-assembled, linear, automated microkeratome.
- Intraoperative visibility.
- Nasal hinge.
- Round-shaped or oval-shaped flap option.
- Several head sizes available for customized flap thicknesses from  $100\mu\text{m}^{(1)}$  to  $140\mu\text{m}^{(2)}$  on average.
- Several Single-Use and metallic suction rings to accommodate for all keratomeries.

## The M2 Single-Use System

- Pre-assembled, rotating, automated microkeratome.
- $360^\circ$  hinge position.
- Several head sizes available for customized flap thicknesses including for Thin-Flap LASIK ( $110\text{-}120\mu\text{m}^{(3)}$  on average).
- Multiple suction rings allow intra-operative customization of flap characteristics.

## The Single-Use Solution

- Protected blade to avoid potential damage.
- Eliminates complications and risks probably linked to damaged or improperly reusable heads.
- Eliminates sterilization and maintenance.
- More rapid patient turnover, greater profitability.
- Lower initial investment costs.

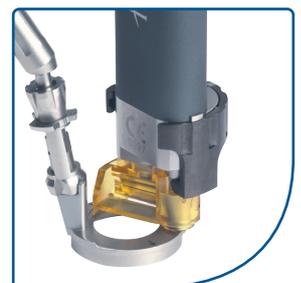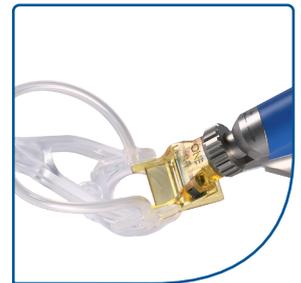

(1) Duffey R.J. Moria One Use-Plus SBK microkeratome: predictably thin, smooth, planar flaps for faster visual recovery. 26<sup>th</sup> ESCRS meeting; Sept 13<sup>rd</sup> 2008; Berlin, Germany

(2) Vryghem J. Single-Use microkeratomies in demand by European surgeons. *Cataract and Refractive Surgery Today Europe* 2007;8:68-70.

(3) Huhtala and al. Corneal flap thickness with the Moria M2 Single-Use head 90 microkeratome. *Acta Ophthalmol. Scand.* 2007;85:401-406.

# Epi-LASIK Surgery


## Epi-K™

The unique design and intelligent features of Moria Epi-K™ have made it the leader in the US epikeratome market<sup>(1)</sup>.

- Epi-LASIK is a refractive procedure performed with an epikeratome which mechanically cleaves the epithelium from the Bowman's membrane, leaving a pristine optical zone for laser ablation. The epithelial flap can then either be discarded or repositioned, according to the surgeon's preference.
- Many surgeons have made Epi-LASIK their procedure of choice for surface ablation. With the Epi-K™ and the latest refinements in technique and postoperative care, Epi-LASIK produces faster healing and visual recovery than all other surface ablation procedures.


## The optimal design for safe, reliable separation time after time

- Handpiece with two independent motors for fast separator oscillation and safe advancement rate.
- Metal separator with proprietary edge geometry specifically for cleaving rather than cutting.
- Disposable plastic head encases each separator for added safety and convenience.
- Unique large applanation plate provides yet an additional margin of safety.
- Several suction rings with adjustable stops to customize the epithelial flap diameter.
- Large diameter suction ring for hyperopes, flat corneas, wavefront guided ablations, and lasers requiring large ablation zones.
- Single-Use suction ring option.

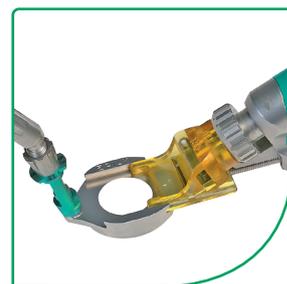

# Corneal Transplant

Superficial Anterior Lamellar Keratoplasty (SALK) ■  
 Deep Anterior Lamellar Keratoplasty (DALK) ■  
 DSAEK and Ultra-thin DSAEK ■

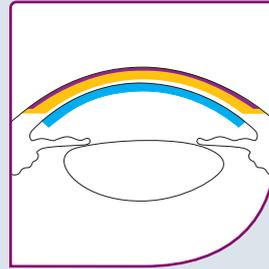



## ALTK-DSAEK Systems

Unique, fully sterilizable equipment for Microkeratome-Assisted Lamellar Keratoplasties, including<sup>(1)</sup>:

- **Descemets' Stripping Automated Endothelial Keratoplasty (DSAEK) and Superficial Anterior Lamellar Keratoplasty Ultra-thin DSAEK** indicated in:
  - Fuchs' and other endothelial dystrophies
  - Post-cataract surgery edema (aphakic or pseudophakic bullous keratopathy)
  - Failed penetrating keratoplasty.
- **Superficial Anterior Lamellar Keratoplasty** for the treatment of superficial corneal opacities resulting from previous refractive surgical procedures, infections, degenerations, dystrophies, superficial scars or trauma.

## Moria offers a full range of sterilizable or disposable devices necessary to perform corneal transplant:

- Automated or manual microkeratomes with multiple heads and suction rings.
- Dedicated instruments, including the Busin's glide and forceps for DSAEK and Ultra-thin DSAEK.
- Hanna punch and ONE® disposable punch.
- ONE® adjustable vacuum trephine.
- Artificial chambers (either reusable or single-use).

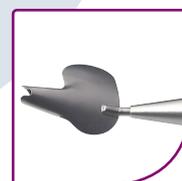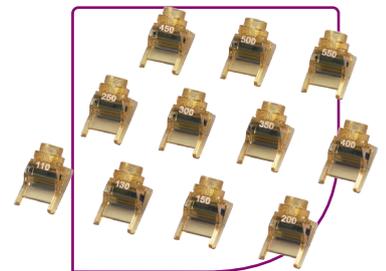

(1) Atlas of Microkeratome Assisted Lamellar Keratoplasty - Massimo Busin, Robert C. Arffa. Slack Incorporated 2006.

# A Common Platform



## Evolution<sup>3E</sup> Control Unit

Operates the One Use-Plus, M2 Single-Use, Epi-K™ and the ALTK-DSA EK Systems .

Moria's console offers a wealth of features to enhance performance, safety, flexibility, and ease-to-use:

- Two high performance pumps maintain safe vacuum.
- "Slow vacuum release" option provides gentle release to minimize potential retinal damage.
- Runs on wall current, with built-in back up battery for uninterrupted use.

### Evolution<sup>3E</sup> Control Unit Technical Specifications:

<b>Dimensions</b>	430 x 240 x 190mm
<b>Weight</b>	13.6 kg
<b>Ambiant temperature</b>	15°C to 35°C (59°F to 95°F)
<b>Non-condensing humidity</b>	45% to 75%
<b>Voltage/cycles</b>	100 V / 50-60Hz 240 V / 50-60Hz
<b>External fuse</b>	500 mA high switching power
<b>Battery capacity and type (inside the control unit)</b>	12 V – 7 Ah (Pb)
<b>Fuse (inside the control unit)</b>	3.15 AT and 500mAT
<b>Inlet : console supply</b>	4-6 bars / 58-87 psi
<b>Outlet: turbine supply</b>	3.2 bars / 46 psi Authorized range : 2.5 to 3.5 bars
<b>Type of protection against electric shock</b>	Safety class BF (according to IEC 60601-1)
<b>Compliance</b>	IEC 60601-1 CE 0120 93/42/EEC ISO 9001 – 2008 version ISO 13485 – 2003 version FDA registered

# Product Specifications

## Microkeratomes Technical Specifications:

Product name	Manual or Automatic	Advance rate	Applanation plate	Cutting mechanism	Oscillations per minute	Flap diameter (mm)	Eye fixation	Head size	Flap thickness	Variable flap orientation
<b>One Use-Plus</b>	Automatic	Adjustable	Yes	Blade in disposable pre-assembled head	15,000 rpm	Customizable	Set of rings according to different keratometries	90 130 Large Cut 110 Large Cut 130	100-110 µm 130-140 µm 110-120 µm 130-140 µm	Nasal
<b>M2 Single-Use</b>	Automatic	Adjustable	Yes	Blade in disposable pre-assembled head	15,000 rpm	Customizable	Set of rings according to different keratometries	90 130	110-120 µm 140-150 µm	360°
<b>Epi-K™</b>	Automatic	Adjustable	Yes	Blunt edge metal separator in disposable pre-assembled head	Confidential	Customizable	Set of rings according to different keratometries	N/A	N/A	Nasal
<b>CB for ALTK-DSA EK</b>	Manual with an autoclavable turbine	Variable	Yes	Blade	15,000 rpm	Customizable	Set of rings according to different keratometries	Full range in reusable or single-use version	Depending on advance speed	N/A
<b>One Use-Plus for Artificial Chamber</b>	Automatic	Adjustable	Yes	Blade in disposable pre-assembled head	15,000 rpm	N/A	N/A	Full range in single-use version	Refer to document #65101	N/A
<b>One Turbine</b>	Manual with an autoclavable turbine	Variable	Yes	Blade in disposable pre-assembled head	15,000 rpm	N/A	N/A	Full range in single-use version	Refer to document #65101	N/A

**Power source:** AC with battery backup  
Regulatory status: 510(k) approved and CE marked



To obtain more information



**MORIA SA**  
15, rue Georges Besse  
92160 Antony  
FRANCE  
Phone: +33 (0) 1 46 74 46 74  
Fax: +33 (0) 1 46 74 46 70  
[www.moria-surgical.com](http://www.moria-surgical.com)

**MORIA JAPAN KK**  
Inami Bldg. 6F  
3-24-2 Hongo Bunkyo-ku  
Tokyo 113-0033  
JAPAN  
Phone: + 81 3 5842 6761  
Fax: + 81 3 5842 6762  
[www.morijapan.com](http://www.morijapan.com)

**Moria Shanghai Office**  
RM4, 10 FL, Kai Li Building  
N°432 Huai Hai West Road  
200052 Shanghai  
CHINA  
Phone: +86 21 5258 5068  
Fax: +86 21 5258 5067  
[www.moria-surgical.com.cn](http://www.moria-surgical.com.cn)

**Moria**  
Leading Innovations in Ophthalmology