Can one-step astigmatism marker

Intended use

Can* pendular marker (#20033) is intended to mark the chosen angle axis on patient's eye(s) in the preoperative time. It is used for positioning toric intraocular lenses.

Accuracy


Accuracy of Can pendular marker is $\pm 1,5^\circ$.

Select the angle


- 1- Unscrew the pendulum half a turn.
- 2- Choose the angle by moving the arrow on the dial.
- 3- Screw the pendulum until it blocks.

Positioning and use of the instrument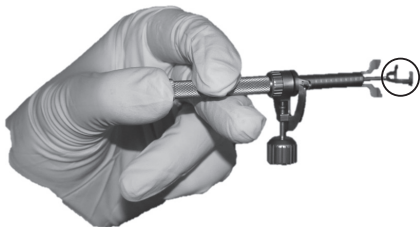

- 1- Put your index and middle fingers on each side of the handle.
- 2- Block the pivot rod with your thumb.
- 3- Put the centering tip on the eye and align it to the limbus of the cornea.
- 4- Move forward the handle so that the axis marker marks the eye.

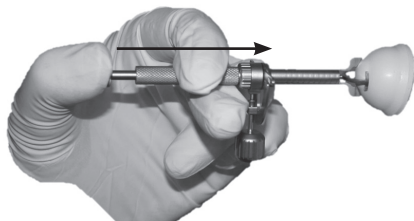Do not angle the centering tip and the axis marker in the same side.

An alternative positioning is to hold the handle with the middle finger and the thumb, and use the index finger for the pivot rod.

Parts of the instrument

- ① Pivot rod
- ② Handle
- ③ Pendulum and dial
- ④ Axis marker
- ⑤ Centering tip

Maintenance

For maintenance, cleaning and sterilization please refer to reusable device instruction #65034. If necessary, the instrument can be disassembled and reassembled as shown below.


Disassembly

STEP 1


Hold the parts ① and ⑤ at the same time and unscrew the part ① by turning it anti-clockwise.

STEP 2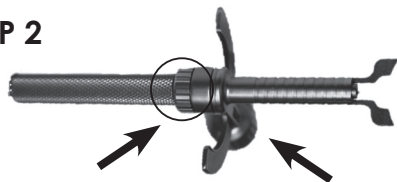

Unscrew the part ② from the part ③.
See surrounded area to unscrew the part ②.

STEP 3

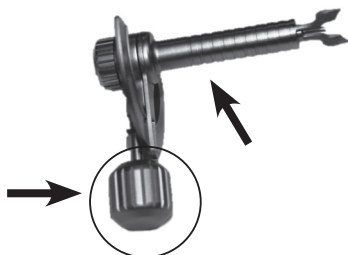Unscrew the pendulum of the part ③ (2 turns only) and disassemble the part ④ from the part ③.

Please note that you have to proceed in the reverse order to reassemble the instrument

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MORIA Inc