EN Instructions for Cleaning, Sterilization, and Maintenance of Moria Composites Ophthalmic Surgical Instruments
CE June 98
CE 0459 (only for measurement instrument)

General Comments

The Composites instrument line has been designed for optimal operation, provided that the recommendations listed in these instructions are followed carefully. The most recent version of this document is available on the MORIA website: <http://www.moria-surgical.com>.

It is the responsibility of the processor to ensure that the processing is performed using proper equipment, materials, and personnel to achieve the desired result. Validation and routine monitoring of the process are required. Any deviations from these instructions by the processor should be validated prior to use for effectiveness and potential adverse consequences.

Description and Intended Use

Moria Composites instruments are reusable handheld devices for performing ophthalmic surgical procedures. The active elements of the forceps, hooks, manipulators, and rings are surgical stainless steel and are attached to handles made from a composite polymer. The use of these instruments is apparent to qualified personnel.

Measurement instruments

Measurement instruments are warranted for half a graduation accuracy, unless otherwise indicated on the instrument, the instruction manual, or its packaging.

Precautions and Warnings

⚠ Moria shall not be liable for any direct or indirect injury experienced by the user or his patients in the event of inappropriate use, care, or maintenance of the instruments.

⚠ Moria disclaims all liability in the event of an accident resulting from repair or modification of the instruments by persons not authorized by Moria.

⚠ The instruments should be used only by physicians and well-trained personnel.

⚠ **Caution USA only: Federal law restricts the use of these devices to physicians or licensed practitioners.** Ensure that the selected instrument is appropriate for the intended medical procedure, is in working order, and that it will not interfere with other medical devices used during the procedure.

⚠ If any instrument is received in defective condition, notify

Moria. Never use damaged or defective instruments. Instruments are delivered non-sterile and must be cleaned and sterilized before use.

⚠ The instruments are supplied in plastic cartridges and, in some cases, with silicone tip protectors. These packaging items are not autoclavable and are not suitable for cleaning, sterilization, or reuse. They should be removed and discarded prior to initial processing.

⚠ Instruments should be immediately cleaned and sterilized after each use.

⚠ Incompatibilities: Do not use bleach.

⚠ For effective sterilization, check that the instruments are entirely clean and free of any residue.

⚠ After sterilization, and prior to each use, inspect instruments under a microscope (minimum 20 times magnification) for changes in physical characteristics that may affect their safe use, effectiveness, or performance. Misaligned forceps tips, bent instruments, dull instruments, and degradation, damage, or loosening of composite material handles indicate that an instrument is at the end of its product life

In particular:

- Discard forceps with misaligned tips.
- Discard hooks, manipulators, rings, and spatulas that are bent.
- Discard instruments that no longer have the requisite sharpness for satisfactory performance.
- Discard instruments in which the metal parts are separated or loose from the composite material handles.
- Discard instruments with degraded or damaged handles.
- Discard instruments with evidence of oxidation.

The above list is not all-inclusive. Sound professional judgment must always be used in determining whether an instrument is in appropriate condition for use.

⚠ In case of doubt, consult the Moria website, the Moria catalogue, or your Moria representative to obtain further information.

Cleaning

- First time processing only: remove the instrument from its plastic cartridge and remove the tip protector. Discard the plastic cartridge which are not autoclavable and are not suitable for reprocessing, or reuse.
- Place the instruments in an open articulated position to facilitate thorough even processing.
- Do not overload too many instruments into the cleaning container and make sure that the active parts do not touch each other during processing.
 - Decontaminate - upon removal from the sterile field, immediately immerse in an enzymatic instrument cleaner (for example: Alkazyne®) in accordance with

the manufacturer's recommendations (e.g., time and temperature).

Note: Refill bath after each use.

- Rinse - with tap water.
- 1st Cleaning –
 - **Manual cleaning:** clean for a minimum of 2 minutes using soft brushes (do not use metallic brushes); or
 - **Mechanical cleaning:** use a mechanical cleaning machine (e.g., ultrasonic cleaning machine) in accordance with the manufacturer's recommendations.
- Rinse - with tap water.
- 2nd Cleaning – proceed in the same manner as the 1st cleaning (use a new bath of enzymatic cleaning solution for each manual cleaning).
- Rinse - with filtered or purified water.
- Dry - with a single-use, lint-free, non-woven, soft cloth; and blow dry as needed.

- Visually inspect the instrument after cleaning. If it is not entirely clean, repeat the previous cleaning steps or dispose of the instrument.

Sterilization

For effective sterilization, check that the instruments are entirely clean and free of any residue. Sterilization with autoclave (steam sterilizer)

Steam sterilization cycle	Preparation	Exposure time ²	Temperature	Minimum drying time, in room chamber ³
Pre-vacuum	wrapped	18 minutes	134°C / 273°F	20 minutes
Pre-vacuum (UK)	wrapped	3 minutes	134°C / 273°F	20 minutes
Pre-vacuum (USA) ¹	wrapped	4 minutes	132°C / 270°F	20 minutes

¹ US only. Note: As per ANSI/AAMI S779
² Sterilization temperature must not exceed 140°C / 284°F
³ Use the autoclave drying cycle to avoid oxidation

Moria recommends the above sterilization processes. Users must validate their sterilization process. The handle made of a composite material may change color after several cleaning and sterilization cycles but this change has no effect on its integrity.

Inactivation of UTA (unconventional transmissible agents) (only for patients at risk of Creutzfeldt-Jakob disease – CJD questionnaire / please refer to French Ministry instruction DGS/R13/2011/449-December 2011)

Storage

- Store instruments in clean dry conditions at a temperature between 15°C and 35°C (59°F and 95°F) and relative humidity between 30% and 75%, in order to prevent risk of condensation inside the packaging.
- Do not store instruments near products or in an environ-

- ment that may have a corrosive or magnetic effect.
- Avoid contact between instruments.

Repairs

These instruments are not intended for repair.

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