



Instructions for use of the One® Single-Use XL Guarded cover #17206DXXX

Indication for use

The Moria ONE® Single-Use XL Guarded cover is designed for performing lamellar keratoplasty. The One® Single-Use XL Guarded cover #17206DXXX is used after using the One® Single-Use Guarded Punch #17207DXXX.

Device components

- Trephine blade housing (see Fig. 1)

To be used with a cutting block base (A) (see Fig. 2), a syringe (C) (see Fig. 2) and a clamp (D) (see Fig. 2), provided separately with the One® Single-Use Guarded Punch #17207DXXX.

The most recent version of this document is available on Moria website (<http://www.moria-surgical.com>) via the restricted access heading «Documentation/Library». Translations are also available in other languages under that same heading. Registration required.





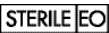






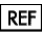


Warning

- These instructions explain only how to use this medical device and do not describe the steps necessary to perform a corneal graft. The surgical procedure remains the responsibility of the surgeon experienced in performing corneal grafts.
- Check the integrity of the pouch.
- Check also the expiration date on the packaging.
- The ONE® Single-Use XL guarded cover is sterile and ready for use. Do not use it if the pouch in which it is packaged is damaged.
- Design of this medical device does not allow its reuse. Reusing this medical device affects its clinical performances and exposes the patient to adverse events.

Directions

- 1 / Check the blade diameter which is stamped on top of the trephine blade housing.
- 2 / Remove the sterile blister from the pouch.
- 3 / Remove the trephine blade housing from the blister.
- 4 / Take the cutting block base and the syringe previously connected with the Luer™ lock (components provided separately with the One® Single-Use Guarded Punch #17207DXXX) with the graft on it and already trephined.
- 5 / Check the integrity of the trephine blade under the microscope.
- 6 / Place the trephine blade housing on the cutting block base, let it lower down by gravity to avoid setting it with a wrong angle (risk of blockage) (Fig. 4) then press down gently to cut the graft (Fig. 5) **Warning : Do not turn the punch during or after trephination.** While holding the assembled punch, grasp the sclera, through the "top lateral" window, with a tooth forceps then gently lift the trephine blade housing up (Fig. 6).
- 7 / Pull gently upward the trephine housing blade from the cutting block base. The donor button should be kept in place on the cutting block base thanks to the aspiration. Then remove completely the trephine housing blade.
- 8 / Disassemble the syringe from the tubing to release the vacuum. The donor button is now ready to be used on the patient.
- 9 / At the end of the surgical procedure, discard the ONE® Single-Use XL Guarded Punch into an appropriate container for destruction (according to the regulations in force).

Explanation of symbols

| | |
|---|---|
|  xxxxxx | CATALOGUE REFERENCE |
|  | QUANTITY |
|  YYYY/MM/DD | USE BY |
|  xxxxxx | BATCH CODE |
|  | STERILIZED USING ETHYLENE OXIDE |
|  | DO NOT REUSE |
|  xxxxxxxx | MANUFACTURER |
|  YYYY/MM/DD | DATE OF MANUFACTURE |
|  xxxxxx | CAUTION: CONSULT ACCOMPANYING DOCUMENT(S) |
|  xxxxxx | CONSULT OPERATING INSTRUCTIONS |
|  | DO NOT USE IF PACKAGE IS DAMAGED |
|  | KEEP DRY |
|  | DISCARD IN A SEPARATE COLLECTION BIN |
|  | CAUTION FOR USA ONLY: US FEDERAL LAWS RESTRICT THIS DEVICE TO SALE BY, OR ON THE ORDER OF, A PHYSICIAN. |

Reference

| | |
|-------------|----------------------------------|
| Reference | #17206DXXX |
| Designation | Single-Use XL Guarded cover DXXX |
| Packaging | 1 per box, sterile, single-use |

Fig.1


#17206DXXX contains the following element



← Trephine blade housing

Fig.2

To be used with #17207DXXX, which contains the following elements:

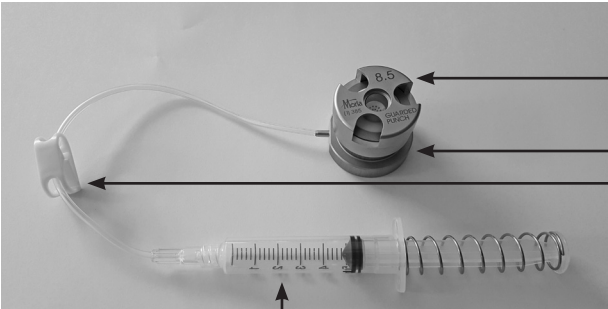

| | |
|---|------------------------|
| A | Cutting block base |
| B | Trephine blade housing |
| C | Syringe |
| D | Clamp |

Fig.3



Fig.4

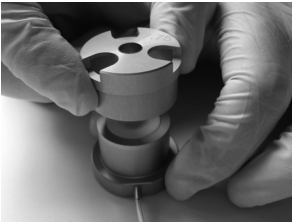


Fig.5


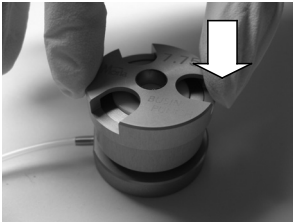


Fig.6

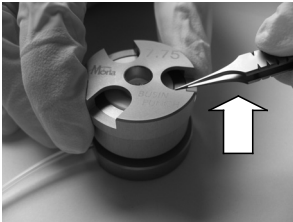

To obtain more information

MORIA SA

27, rue du Pied de Fourche
03160 Bourbon l'Archambault
FRANCE
Phone: +33 (0) 4 70 67 09 04
Fax: +33 (0) 4 70 67 06 61
www.moria-surgical.com



MORIA Inc

1050 Cross Keys Drive
Doylestown, PA 18902
USA
Phone: (800) 441 1314
Fax: + 1 (215) 230 7670
www.moria-surgical.com

Moria Japan K.K.

Arcadia Building 6F
1-12-3 Kanda
SudachoChiyoda-Ku
Tokyo 101-0041
JAPAN
Phone: 81-3-6260-8309
Fax: 81-3-6260-8310
www.moriajapan.com

Moria Shanghai Office

RM4, 10 FL, Kai Li Building
N°432 Huai Hai West Road
200052 Shanghai
CHINA
Phone: +86 21 52585278
Fax: +86 21 52585278
www.moria-surgical.com

Moria