Cleaning protocol in the cleaning space for Composites Squeeze Forceps

Tap Water rinsing Step 1

Minimum 30 seconds

Manual cleaning Step 2

Immersion in **neutral Ph cleaning solution***

Remove protector tip
Activate forceps mechanism 5 times

Clean body of instrument with soft brush **without touching tube or active parts**, until all visible soil has been removed

Lumen for cleaning

With the **5ml syringe and special cannula, flush cleaning solution 5 times per lumen**

Tap Water rinsing Step 3

Minimum 30 seconds

US Bath Cleaning Step 4

US bath with fresh neutral Ph cleaning solution*
Sonificate for 5 mn

Tap Water rinsing Step 5

Minimum 30 seconds

Rinsing Step 6


With the 5ml syringe and special cannula, **flush demineralized water***


Immersion in **demineralized water***

How to remove and put the protector tip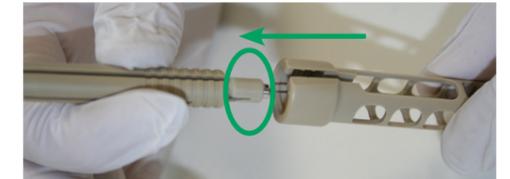

To remove the protector, squeeze and move back the forceps till the head (circled on the photo) is out.

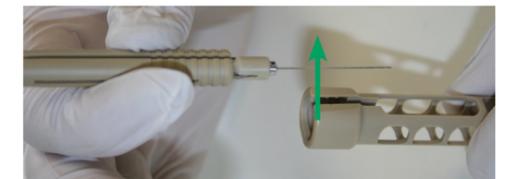Remove the forceps using the opening **without touching it with the active parts**. Proceed in reverse order to replace the protector.

WARNING When putting back the protector check that the forceps is well positioned

Drying Step 7

With the **5 ml syringe and special cannula**, flush air 5 times per lumen

Wipe instrument with a non woven lint free disposable cloth

Dry with **filtered dry air**

Tools

#19149
Cleaning brush

Syringe luer lock 5 ml
(a bigger size will make the flushing difficult)

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