This instrument is designed for intraocular use. FORCEPS are designed to grasp, and manipulate tissues during a surgical intervention. . SCISSORS are designed to cut segment tissue during eye surgery.

CAUTION: FEDERAL (U.S.A) LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN

- Release the Dex Tip™ from pouches and ensure sterile transfer to the sterile field.
- Twist Dex Tip™ onto Katalyst Part **CV-Handle or CVO-Handle** until it stops.
- Remove protective tube.
- The instrument is ready for surgical use.
- Squeeze handle to push together the tips

REF	Type	Y = Gauge
CVF4XXX-2Y	Forceps Tips	20ga, 23ga, 25ga, 27ga 0.81mm, 0.64mm, 0.5mm, 0.41mm
CVS5XXX-2Y	Scissors Tips	
CV-Handle	Reusable Handle	
CVO-Handle	Reusable Handle	

- Various tip designs allow for different surgical applications.
- Dex Tips™ sterilized by Ethylene Oxide, single-use, latex-free.

Clean and rinse the instrument immediately after each use. Failure to clean promptly may result in adherent particles that may resist cleaning and complicate or resist future sterilization. CV-Handle and CVO-Handle must be completely cleaned and rinsed of all foreign matter. Proper cleaning and maintenance procedures will ensure correct function and extend the useful life of the instrument. The max life will be achieved if the instrument is stored in its proper case until the surgeon asks for it and is returned to its case immediately after flushing on the field, preferably with sterile water.

1. Remove blood, saline, tissue and other contaminants immediately by flushing with distilled/deionized water if necessary.
2. Clean instrument with mild soap
 - a. Do not use corrosive cleaning agents (i.e. bleach). Cleaning solutions and rinses at or near a natural pH (7.0) are best.
 - b. Do not use abrasive cleaners
 - c. Enzymatic cleaners should be used to remove protein deposits. If used, follow the enzymatic cleaner's instructions and rinse thoroughly.
3. Dry instrument completely and carefully with compressed air.
4. Place instrument into Sterilization Tray.

- Sterilize instrument according to hospital guidelines.
- Steam autoclaving has been validated in the wrapped configuration and can be sterilized wrapped or unwrapped.
 - Standard autoclave cycle: Steam sterilize at 270°F (132°C) for 4 minutes or greater. Other time and steam temperature cycles may also be used. However, user must validate any deviation from recommended time and temperature.
 - 20 minutes of dry time has been validated and was determined to be sufficient to completely dry the instrument.
 - Contact the manufacturer of your steam autoclave to confirm appropriate temperatures and sterilization times.
 - Autoclave temperatures should not exceed 280°F (137°C); handle, insulation or other non-metallic parts may be damaged.
- Use of plasma sterilization will compromise instrument life, is not recommended and will void the warranty.

This device is to be used only by qualified physicians.
Follow medical operating procedures during surgery.
This product is intended for single use, one patient only.
Reuse or re-sterilization may cause diminished product performance, contamination or infection.

Impugnatura CV e Impugnatura CVO e Dex Tips™ (forcipe e forbici) ITALIAN
STERILI, MONOUSO

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Katalyst Surgical, LLC 754 Goddard Ave. Chesterfield, MO USA 63005