**Dex™ (Forceps and Scissors)
STERILE, SINGLE USE**

ENGLISH

DESCRIPTION/INDICATION FOR USE (by physicians in the operating room)

This instrument is designed for intraocular use. FORCEPS are designed to grasp, and manipulate tissues during a surgical intervention. . SCISSORS are designed to cut segment tissue during eye surgery.

CAUTION: FEDERAL (U.S.A) LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN

INSTRUCTIONS FOR USE AND EXPIRATION DATE:

- Release the instrument from pouches and ensure sterile transfer to the sterile field.
- Remove protective tube.
- The instrument is ready for surgical use.
- Squeeze basket handle to push together the tips

REORDER CODES: (DVF4XXX-2Y, DVS5XXX-2Y)

REF	Type	Y = Gauge
DVF4XXX-2Y	Forceps	20ga, 23ga, 25ga, 27ga
DVS5XXX-2Y	Scissors	

FEATURES:

- Various tip designs allow for different surgical applications.
- Sterilized by Ethylene Oxide, single-use, latex-free.

CAUTION:

This device is to be used only by qualified physicians.
Follow medical operating procedures during surgery.
This product is intended for single use, one patient only.
Reuse or re-sterilization may cause diminished product performance, contamination or infection.

PRECAUTION:

Do not use if package is open, damaged or wet.
Do not use if expiration date shown on the label is exceeded.

ADVERSE REACTIONS:

There are no adverse reactions associated with the use of this product other than those related to the specific procedure being performed (to be assessed by the physician).

WARNINGS:

Do not use this device as a tamponade, unless specifically indicated.
Do not use this device for foreign body removal, unless specifically indicated.

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Katalyst Surgical, LLC 754 Goddard Ave. Chesterfield, MO USA 63005