HOLEKAMP DRIP DROPPER, DISPOSABLE BACKFLUSH AND DISPOSABLE CANNULAS
DESCRIPTION/INDICAZIONE FOR USE (by physicians in the operating room)

This cannula is designed to be inserted into the eye to facilitate simultaneous irrigation and aspiration.

CAUTION: FEDERAL (U.S.A) LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN

INSTRUCTIONS FOR USE:

- Release the cannula from pouches and ensure sterile transfer to the sterile field.
- Remove protective tube.
- If present, advance the T-tube forward over the silicone tip
- Squeeze the bulb to evacuate the air and submerge tip into intended fluid. Release bulb to aspirate fluid into cannula and bulb.
- Hold the tip upright to keep the cannula clear of air.
- Insert Cannula tip into eye and slowly squeeze bulb to inject fluid.
- Disposable Backflush Handpiece:**
 - Release the cannula from pouches and ensure sterile transfer to the sterile field.
 - Remove protective tube.
 - If present, advance the T-tube forward over the silicone tip
 - The instrument is ready for surgical use.
 - For Active Backflush capacity, remove leur from back of handle and hook to aspiration
 - Prime the handpiece before use
- Disposable Cannulas:**
 - Release the cannula from pouches and ensure sterile transfer to the sterile field.
 - Remove protective tube.
 - If present, advance the T-tube forward over the silicone tip.
 - Attach cannula to a reusable backflush handpiece using leur connection

REF	Description	Reflux Bulb
DV1200-2XX	Soft Tip Drip Dropper	“F”= Fluted bulb No “F” = not Fluted bulb
DV1214-2XX	Diamond Dusted Soft Tip Drip Dropper	
DVC1700-2XX	Blunt Tip Drip Dropper	
DVC1400-2X	Blunt Tip Backflush	
DVC1410-2X	Soft Tip Backflush	
DVC1412-2X	Brush Tip Backflush	
DVC1414-2X	Diamond Dusted Backflush	
DVC1300-2X	Soft Tip Disposable Cannula	
DVC1314-2X	Diamond Dusted Disposable Cannula	
DVC1325-2X	Brush Tip Disposable Cannula	
DVC1350-2X	Blunt Tip Disposable Cannula	

- FEATURES:
- Various tips allow for different surgical applications.
 - Sterilized by Ethylene Oxide, single-use.

CAUTION:
This device is to be used only by qualified physicians.
Follow medical operating procedures during surgery.
This product is intended for single use, one patient only.
Reuse or re-sterilization may cause diminished product performance, contamination or infection.

PRECAUTION:
Do not use if package is open, damaged or wet.
Do not use if expiration date shown on the label is exceeded.
Do not use or discontinue use if debris is seen.

ADVERSE REACTIONS:
There are no adverse reactions associated with the use of this product other than those related to the specific procedure being performed (to be assessed by the physician).
POSSIBLE COMPLICATIONS:
Hypotony/Collapse of the eye during active backflush
Damage to retina

WARNINGS:
There are no warnings associated with the use of this product other than those related to the specific procedure being performed (to be assessed by the physician).

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Katalyst Surgical, LLC, 754 Goddard Ave.