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[www.moria-surgical.com](http://www.moria-surgical.com)

## INSTRUMENT CARE, CLEANING AND STERILISATION OF REUSABLE AND AUTOCLAVABLE INSTRUMENTS

### CE June 98 CEO459

(only for measuring instruments)

**Note: For details on measurement instruments, please see the information on the packaging.**

This notice in this document is printed in French, English, Italian, Spanish and Dutch. Each language version can also be published in large format on the Moria website ([www.moria-surgical.com](http://www.moria-surgical.com) – restricted access, 'library' heading). German, Arab, Swedish, Finnish, Danish, Portuguese and Polish versions are available in electronic form on the Moria website. Please note printed versions are available upon request.

#### Measurement instruments

Measurement instruments are warranted for a +/- half a graduation tolerance, unless otherwise indicated on the instrument, the instruction manual, or its packaging.

#### ATTENTION:

⚠ Moria shall not be liable for any direct or indirect injury suffered by the customer in the event of the inappropriate use, care, cleaning or sterilisation of the medical device.

⚠ **Moria disclaims all liability in the event of an accident linked to intervention on the instrument by a person or entity not authorised by Moria.**

The Moria warranty does not apply in the event of any disassembly, modification or intervention on the instrument (e.g.: marking) by a person not authorised by Moria (if the instrument requires repair it must be decontaminated before being returned, otherwise Moria reserves the right to return the instrument in the same state that it was received).

#### Products concerned:

Range of Moria reusable medical devices compatible with autoclave (134°C/273°F steam steriliser) and soda. Please refer to the Moria instrument catalogue for incompatibilities (2011 version).

#### Incompatibilities:

- Do not use bleach.
- Do not use soda for instruments made partly of aluminium alloy (see Moria instrument catalogue 2011 version available on the website).

#### Storage conditions

- Store instruments in clean dry conditions (temperature between 15°C/59°F and 35°C/95°F and relative humidity between 30 and 75%) in order to prevent risk of condensation inside the packaging.
- Keep the protective cover or the Moria protective tip on the active part of the instrument during storage.
- Do not store instruments near products or in an environment that may have a corrosive or magnetic effect.
- Avoid contact between instruments especially if they are made of different materials.
- Instruments made of titanium are identified by the letters "TI" in the product reference.

#### Inspection before every use

- Instruments that are not delivered sterile must be cleaned and sterilised before use.
- Make sure that the selected instrument is appropriate, is in working order, and that it will not interfere with other medical devices used during the medical procedure.
- Make sure that the instrument has not been subjected to impact and that it is free of marks or signs of wear, primarily the parts in contact with the patient or with medical staff.
- For catheters connected to another medical device, check that connections are secure (Luer Lock) and sealed tight.
- In case of doubt, consult the Moria instrument catalogue, the Moria website or your Moria agent to obtain further information on indications and restrictions of the use of the medical device.

#### Sterilization

##### Preliminary precautions

- Process the instruments in open articulated position, without unclipping at the rear, to promote through even processing.
- Make sure that the instruments are positioned correctly

without overloading and that the active parts do not touch each other during the process.

- For hollow medical instruments, draw in and expel the disinfectant fluid several times making sure that no foreign substances remain. Regularly check for the absence of organic or metallic debris/deposits under a microscope.

- If necessary, ensure that the Moria protective tip is properly positioned and it accompanies the instrument during all stages of cleaning / disinfection / sterilization.

##### Protocol for cleaning the medical device

Refer to cleaning protocol IIP001 available from the Moria website ([www.moria-surgical.com](http://www.moria-surgical.com) – restricted access, 'library' heading) which recommends double cleaning:

- Immerse immediately upon removing from the sterile field in a non-aldehyde disinfectant-detergent solution for 15 minutes at 20°C/68°F in accordance with the manufacturer's recommendations.
- 1<sup>st</sup> manual cleaning: Mechanical cleaning for a minimum 2 minutes using soft brushes (metallic brushes prohibited), or using an ultrasonic machine (provided that the machine does not recycle the cleaning products).
- Rinse with tap water.
- Immerse in a non-aldehyde disinfectant-detergent solution (different bath from the previous).
- 2<sup>nd</sup> manual cleaning: Proceed in the same way as for the 1<sup>st</sup> manual cleaning.
- Rinse with filtered or purified water.
- Dry the device with a single-use, lint-free non-woven soft cloth.

For effective sterilization, check that the instruments are entirely clean and free of any residue. Sterilization with autoclave (steam sterilizer).

Steam sterilization cycle	Preparation	Exposure time	Temperature <sup>1</sup>	Minimum drying time, in room chamber <sup>2</sup>
Pre-vacuum	wrapped	18 minutes	134°C / 273°F	20 minutes
Pre-vacuum (UK)	wrapped	3 minutes	134°C / 273°F	20 minutes
Pre-vacuum (USA) <sup>3</sup>	wrapped	4 minutes	132°C / 270°F	20 minutes

<sup>1</sup> US only, Note: As per ANSI/AAMI ST79:2010 A1 and A2:2011.

<sup>2</sup> Sterilization temperature must not exceed 140°C/284°F.

<sup>3</sup> Use the autoclave drying cycle to avoid oxidation.

Other cycles and temperatures can be used (to be approved by the user). However, autoclave sterilization temperature must not exceed 140°C/284°F.

#### Sterilization only in the United States of America

- Products concerned: range of Moria reusable medical devices compatible with autoclave, with the exception of Moria Hanna Trepphine, ref. 17166 (refer to Hanna Trepphine User Manual, ref. 65022).
- Autoclave, steam sterilization by pulsed pre-vacuum process, at a minimum of 132°C/270°F (not to exceed 140°C/284°F) for 4 minutes minimum.

Inactivation of UTA (unconventional transmissible agents) (only for patients at risk of Creutzfeld-Jakob disease – CJD questionnaire / please refer to French Ministry guide DGS/R13/2011/449 – December 2011).

Moria recommends the above sterilization processes. The use of a cleaning and/or sterilization protocol, other than the one recommended by Moria, will be the responsibility of the end user. Users must validate their sterilization process.

Inactivation of UTA (unconventional transmissible agents) (only for patients at risk of Creutzfeld-Jakob disease – CJD questionnaire / please refer to French Ministry instruction. DGS/R13/2011/449-December 2011).

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