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Posterior Segment Micro-Incision Surgery

IMPORTANT

FOLLOWING THESE RECOMMENDATIONS WILL HELP YOU MAINTAIN THE QUALITY OF YOUR INSTRUMENT. IMPROPER MAINTENANCE AND STORAGE WILL DAMAGE YOUR INSTRUMENT.

Information sheet for MORIA distally controlled instruments, for posterior segment micro-incision surgery

- 1 Instruments are delivered with a TEFLON®/silicone protector. This protector should be kept on the instrument during storage and sterilization. It should be taken off carefully with the instrument **in retracted** position and should be put back immediately after cleaning.

- 2 Instruments are all marked with an item number on the handle. The last three digits of the item number are also engraved on the cone of the active part of the instrument (For example: 099 for #19099/23 and 004 for #20004). **These three digits must always match the last three digits engraved on the handle.** Handles are not interchangeable.

- 3 Follow the recommendations of the cleaning protocol #17IPT01. Immediately after each surgery, the instrument must be disinfected, cleaned, and rinsed in the following manner :

- **disinfection:** immediate immersion of the instrument in an aldehyde-free detergent and disinfectant solution (immersion duration in accordance with the manufacturer's recommendations).

- **cleaning and rinsing:**

- unscrew the end of the instrument,

- screw the cleaning connector (provided with the instrument) onto the active tip of the instrument,

- with a syringe, inject distilled water via the Luer lock,

- clean and rinse the aspiration silicone tubing, if available,
- dry the set with filtered compressed air,
- screw the handle back onto the active part tip of the instrument; making sure to screw on the correct handle by matching the item numbers, as mentioned above, immediately replace the TEFLON®/silicone protector.

Cleaning, disinfection and sterilization protocols are available in the Member Area of the Moria website at www.moria-surgical.com. Username and Password required to obtain access.

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MORIA Inc
Moria Commercial (China) Co. Ltd