# Moria

Ophthalmic Instruments\*

\*Instruments Ophthalmologiques

#19336/XX  
#19337/XX  
#19338/90  
#19354/XX

## One Use-Plus

### EN Set up of the OUP (One Use-Plus) and OUP LC (One Use-Plus Large Cut) set

#### Indication for use

OUP and OUP LC heads are indicated for the creation of lamellar nasal-hinged flaps during refractive surgery when used in combination with the OUP handpiece by Moria.

#### Device components

- OUP or OUP LC head with blade (1)
- Suction ring with vacuum tubing (2)
- Key for the adjustable stop pin (3)

#### Warning

- This instruction for use only refers to the use of medical device but does not describe surgical steps to perform refractive surgery. Refractive surgery must be performed only by experienced corneal surgeons.
- Check the integrity of the pouch.
- Check the expiration date on the labels.
- Inspect the blade under a microscope.
- The device is sterile and ready for use. Do not use it if the pouch in which it is packaged is damaged.
- **The reuse of single-use products, or the use of consumables other than those supplied by MORIA, may entail serious surgical consequences for the patient and damage the device.**
- Do not mix components from different single-use packs (head and ring).
- Do not use OUP LC head (head labelling: "110L" or "130L") with OUP metal rings (#19391/xx).
- **Design of this medical device does not allow its reuse. Reusing this medical device affects its clinical performances and exposes the patient to adverse events.**

Instructions for use and any updates are available on the [moria-surgical.com](http://www.moria-surgical.com) website.

The most recent version of this document is available at <http://www.moria-surgical.com> via the restricted access heading "Documentation/Library". Translations are also available in other languages under that same heading. Registration required.

#### Directions

- 1 / Open the pack by passing your finger through the opened corners.
- 2 / Screw the locking screw clockwise to fix the head on the handpiece. Check that the head is screwed on tightly to the handpiece. The head should not be able to move.
- 3 / Once assembled, inspect the blade under a microscope.
- 4 / Prior to use, lubricate the guiding tracks of the suction ring (4) with a physiological saline solution.
- 5 / Immerse the head (5) in a physiological saline solution until the immersion limit (6), then active the handpiece during 5 seconds. Check that the blade (7) oscillates freely.
- 6 / After the procedure, disassemble the head and ring, put them back into the blister and discard everything in a biohazard container.

#### One Use-Plus Large Cut (OUP LC)

##### Performance :

The corneal flap diameter obtained with the OUP LC is 0.5 to 0.6mm larger than that obtained on the same eye with the OUP.

##### Indications :

Moria recommends not to use the OUP LC on corneas with average keratometry equal or greater than 44.5D and/or corneal diameter equal or under 11mm.

##### Nomogram :

The OUP LC allows the creation of a larger ablation zone than the OUP. To achieve this, the stop value selected should be 0.25 to 0.5 greater than that one selected with the OUP -1, according to the average keratometry.

Refer to the ONE USE-Plus nomogram (#65041).

##### Reminder regarding the nomogram :

The nomogram provided by Moria is a guideline only, and the results obtained may differ from those indicated in the nomogram. A variety of factors has an influence on the surgical result:

- Intra-ocular pressure
- Degree of myopia
- Age
- Gender
- Ethnicity
- Corneal hydration
- Suction time
- Pre-operative topography
- Surgical technique

Moria and its partners do not recommend their own nomograms based on analysis of their post-operative datas.

## List of references

Reference	Description	Packaging
19336/90 19336/130	Pack of one sterile 90/130-micron disposable OUP calibrated head and one sterile "-1" disposable OUP suction ring	1 per box / Sterile / Single-use
19337/90 19337/130	Pack of one sterile 90/130-micron disposable OUP calibrated head and one sterile "0" disposable OUP suction ring	
19338/90	Pack of one sterile 90-micron disposable OUP calibrated head and one sterile "+1" disposable OUP suction ring	
19354/110 19354/130	Pack of one sterile 110/130-micron disposable OUP LC calibrated head and one sterile "-1 LC" disposable OUP LC suction ring	

## Material used in combination with

Reference	Packaging
19345	OUP handpiece (blue color)
19345C	Wrench
22519514	Storage box for the OUP handpiece
19360	Evolution 3 console
19380	Evolution 3E console
19361	Footswitch
19381	Footswitch Epi-K™
19138	Aspiration tubing
19149	Disposable brush

## User information

Reference	Packaging
65039	One Use-Plus single-use plastic ring user manual
65041	One Use-Plus single-use plastic ring nomogram
65038	EVOLUTION 3 console user manual
65051	EVOLUTION 3E console user manual (serial number below 5000)
65060/INTL	EVOLUTION 3E console user manual (serial number 5000 and above)
65073	Annexe "Guidance and manufacturer's declaration: electromagnetic emissions and immunity"

# Notice de montage du set OUP (One Use-Plus) et du set OUP LC (One Use-Plus Large Cut)

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**MORIA Inc**